

# Instruction for Use

## Formative Liquid (Accessory for Dental Ceramic)



## Instruction for Use

【Device Name】 Formative Liquid

【Model and Specifications】

Category	Code	Spec(ml/bottle)
Opaque Liquid	OF	15, 50, 240
Modeling Liquid	CFS	15, 50, 240
Glaze & Stain Liquid	GY	5, 15, 50, 240

【Ingredients】 The formative liquid made by Pure water (H<sub>2</sub>O), 1,3 butanediol (C<sub>4</sub>H<sub>10</sub>O<sub>2</sub>) and 1,5 pentanediol (C<sub>5</sub>H<sub>12</sub>O<sub>2</sub>).

【Intended Use】 Formative liquid divided into modeling liquid, opaque liquid, glaze & stain liquid. They are used to be modeled the dental ceramic for making porcelain veneers.

【Indications】 Accessory for mixing Dental Ceramic porcelain powder indicated for tooth damage (decayed tooth, damaged tooth, etc.), fixed-bridge restorations for tooth absence, aesthetic restoration for tooth discoloration. Formative liquid avoids rapid drying of the ceramic material and also causes increased plasticity when layering. Recommended Powder and Liquid at different proportion as below:

Model	Porcelain Powder	Formative Liquid	Recommended Powder and Liquid mixing proportion
Metal Ceramic	BOND (WO)	Opaque liquid (OF)	1g : 0.67g
	Opaque	Opaque liquid (OF)	1g : 0.43g
	Opacious Dentine, Dentine, Semi-transparent, Transparent	Modeling Liquid (CFS)	1g : 0.39g
	Glaze, Corrective, Stain	Glaze & Stain Liquid (GY)	1g : 0.67g
Zirconia Ceramic	Opacious Dentine, Dentine, Semi-transparent, Transparent	Modeling Liquid (CFS)	1g : 0.39g
	Glaze, Corrective, Stain	Glaze & Stain Liquid (GY)	1g : 0.67g

【Contraindication】 Allergic reactions to the ingredients in this product.

【Precautions】

- For dental use only and keep it clean.
- Can't put into mouth. If accidental contact, flush immediately with large amounts of water. Consulting with a doctor in case of allergy, abdominal pain and diarrhea are strongly recommended.

【Instructions】 The product is used together with dental ceramic.

- Modeling the Bond (Wash opaque), Opaque with the Opaque liquid (OF).
- Modeling the Opacious Dentine, Dentine, Semi-transparent, Transparent with the Modeling liquid (CFS).
- Modeling the Glaze, Corrective, Stain with the Glaze & Stain liquid (GY).
- Mix the ceramic powder with the corresponding liquid until paste and then following the work instruction and firing parameters to operate.
- Avoid vigorous mixing which will tend to incorporate air bubbles with the paste and, both during

and after mixing, examine for compliance with Uniformity and be free from extraneous materials by visual inspection.

**【Storage】** Do store at proper temperature or keep out of intense light. Don't keep with humidity. Keep the product out of the reach of the Children. Package tight closed.

**【Transportation】** This product is not regulated for transport of dangerous goods. Examine whether the package of the containers are integrate and tighten closed before transport. No divulgence, no collapse, no precipitation or no damage during the course of transportation. Don't put the goods together with strong base, water and so on. During transport should prevent exposure, rain and high temperature.

**【Dispose】** This product is not regarded as hazardous waste. When dispose this product or its container refer to National or Local regulations before handing.

**【Date of manufacturing】** See the labeling.

**【Shelf Life】** 12 months.



**BAOT Biological Technology Co., Ltd**

Unit 1 First Floor, Second Floor and Unit 3 Fourth Floor, No. 12 Building, 106 Qihao Road, Torch Development District Zhongshan Guangdong 528437, China

Tel: +86-760-87893825 Email: [baotw@baot.biz](mailto:baotw@baot.biz)



**Shanghai International Holding Corp. GmbH (Europe)**

Eiffestrasse 80, 20537 Hamburg, Germany

SRN: DE-AR-000000001



**Umedwings Netherlands B.V.**

Treubstraat 1,2288EG,Rijswijk, The Netherlands

SRN:NL-IM-00000454

**【Symbol】**

Symbol	Instruction	Symbol	Instruction
	Fragile, handle with care.		Temperature limitation - 18 °C ~ 50 °C
	Consult instructions for use, or consult electronic instructions for use(eIFU)		Humidity limitation < 80%
	Keep dry		Do not use when the packing is damaged
	Attention(See user's manual)		Stacking limit by 5 layers
	Date of manufacturing		Prevent from heat
	Date of expiry		Prescription Use (Rx)
	Lot number		Manufacturer and address
	Conformity to the requirements of the applicable EC directives		Authorized Representative in the European Community/European Union
	Medical device, indicates the item is a medical device		Importer